# TRACHEOCUR® COMPRESSES

D

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EN

**Purpose:** TRACHEOCUR® COMPRESSES are used as mechanical barriers, to absorb tracheal secretions and for compression. They are worn under the tracheal cannula, keep the tracheostoma and clothing dry, and form a barrier between the tracheal cannula and the skin. TRACHEOCUR® COMPRESSES are available in various versions and different absorption levels.

## **Application:**

Fig. 1) SL TRACHEOCUR® COMPRESS

Fig. 2) TRACHEOCUR® COMPRESS

Fig. 3) AL TRACHEOCUR® COMPRESS

Through a cut at the opening, the slit TRACHEOCUR® COMPRESS can be used without removing the tracheal cannula (Fig. 1). With the perforated TRACHEOCUR® COMPRESS, the tracheal cannula is inserted through the hole before use (Fig. 2). The metal-coated TRACHEOCUR® COMPRESSES are to be placed with the metal coating towards the skin (Fig. 3 and 2).

**Cleaning:** TRACHEOCUR® COMPRESSES are intended for one-time use. If TRACHEOCUR® COMPRESSES are wet, they should be replaced for hygienic reasons. Replacement should take place at the latest every time the cannula is changed.

**Safety notices:** There is a risk of infection in case of excessively long use. When using the TRACHEOCUR® COMPRESSES, the tracheal cannula must be worn with a cannula strap. Abrasion marks and the sliding out of the tracheal cannula are thus prevented.

**Product disposal:** The products may only be disposed of in a germ-free state according to the applicable national regulations for waste products.

Subject to product amendments by the manufacturer at any time.

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BMV® Bender Medical Vertrieb GmbH · 53804 Much · Germany · [www.bmv-med.de](http://www.bmv-med.de)