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# Directions for use for TRACHEOCUR® STOMA SEAL

## TRACHEOCUR® STOMA SEAL

Item No.: T137040

The directions for use are intended to inform the doctor, nursing staff and patients/users about correct handling.

Please read carefully before use!

### INDICATION:

TRACHEOCUR® STOMA SEAL is used to seal an uneven tracheostoma in conjunction with a TRACHEOCUR® tracheostoma plaster.

### APPLICATION:

TRACHEOCUR® STOMA SEAL is always used with a TRACHEOCUR® BASE. Before application, the protective film is removed on the adhesive side (see Figure 1). TRACHEOCUR® STOMA SEAL is affixed with the adhesive side on the tracheostoma (see Figure 2).

In order to reliably seal the tracheostoma, it may be necessary, depending on the unevenness of the tracheostoma, to affix a second TRACHEOCUR® STOMA SEAL on top of the first. The plaster is affixed over the affixed TRACHEOCUR® STOMA SEAL in such a way that the opening is not closed (see Figure 3).

TRACHEOCUR® STOMA SEAL is intended for single use. For hygienic reasons, TRACHEOCUR® STOMA SEAL must be replaced daily. If the TRACHEOCUR® STOMA SEAL is contaminated with secretions, an earlier replacement may be necessary.

### CLEANING:

TRACHEOCUR® STOMA SEAL cannot be reprocessed for hygienic reasons.

### SAFETY NOTICE:

There is a risk of infection with repeated use of TRACHEOCUR® STOMA SEAL and exceeding of the maximum wearing time. There are not currently any known allergies.

### DISPOSAL:

The product must only be disposed of according to the applicable national regulations for waste products.

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BMV® Bender Medical Vertrieb GmbH