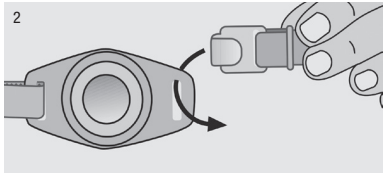
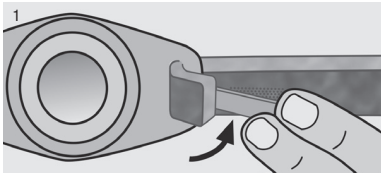
**FORLIFE**  
BERLIN



**TRACHEOCUR<sup>®</sup>**

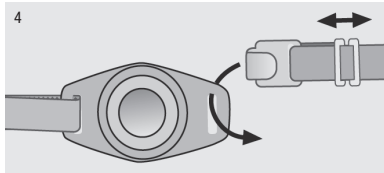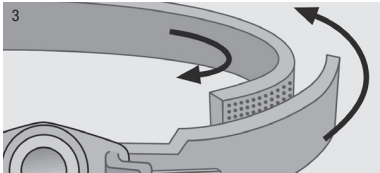**TUBEHOLDER**

## Hinweise für die Anwendung/Notes on application/Avvertenze per l'uso/Gebruiksaanwijzing

## Hinweise für die Anwendung/Notes on application/Avvertenze per l'uso/Gebruiksaanwijzing

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**Purpose** TRACHEOCUR® TUBE HOLDERS are used to fit and secure a tracheal cannula in the tracheostoma.

**Application** TRACHEOCUR® TUBE HOLDERS are available as one- and two-part cannula straps and in versions with Velcro or hook fastenings. TRACHEOCUR® TUBE HOLDERS with a Velcro fastening are particularly well-suited for use on recumbent patients and during the night. TRACHEOCUR® TUBE HOLDERS should not be worn for longer than 24 hours. TRACHEOCUR® TUBE HOLDERS can be easily attached to the tracheal cannula with the Velcro fastening (Image 1) or the hook (Image 2) and can be opened quickly in an emergency. The neck size can be adapted individually for the two-part cannula straps using a Velcro fastening at the wide section of the strap (Image 3). For the TRACHEOCUR® VARIO and the TRACHEOCUR® COMFORT, the neck size is adjusted according to Image 4. For the TRACHEOCUR® BASIC TAPE, the neck size can be adjusted using the Velcro fastening with which the strap is attached at the cannula flange (Image 1). The

correct positioning of the cannula strap in the cannula flange is to be ensured. The tracheal cannula must sit securely in the tracheostoma. TRACHEOCUR® TUBE HOLDERS are a single-patient product.

**Safety notice** There is a risk of infection if the cannula strap is not changed in the specified period. If the cannula strap is too tight, it can lead to pressure points on the neck. If the tracheal cannula does not sit correctly due to incorrect fitting of the cannula strap, pressure stenosis can occur in the trachea.

**Product disposal** The products may only be disposed of in a germ-free state according to the applicable national regulations for waste products.

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BMV® Bender Medical Vertrieb GmbH · 53804 Much · Germany · [www.bmv-med.de](http://www.bmv-med.de)